

TECH MANUAL

DANNINGER

500 & 520

500

TABLE OF CONTENTS

I. OPERATION

| | |
|--|-----------|
| A. Product Description | pg. 1 |
| B. Safety Features | pg. 2 |
| C. Unpacking Unit | pg. 2 |
| D. Set Up | pg. 3 |
| E. Operating the CPM | pg. 3-4 |
| F. Step by Step | pg. 4-5 |
| G. Step by Step -- Neuro-Muscular Electrical Stimulation | pg. 5 |
| H. Muscle-Stim Instruction | pg. 6-7 |
| I. Use of Patient Control Switch | pg. 8 |
| J. Memory Features | pg. 8 |
| K. Bilateral Application | pg. 8-9 |
| L. Attachment of the Patient Kid (Soft Goods) to Unit | pg. 9 |
| M. Measuring Patient and Adjusting Length of Unit | pg. 9-10 |
| N. Attaching the CPM System to the Bed | pg. 10-11 |
| O. Recommended Care of the CPM Unit | pg. 11 |
| P. Accessories/Cosmetic Parts Ordering Information | pg. 12 |

II. SERVICE

- A. Safety Precautions when Servicing the DANNIFLEX™ 500 CPM System pg. 13-15
 - 1. Continuous Passive Motion (CPM) Machine Safety Precautions
 - 2. Explanation of Warnings Specific to the DANNIFLEX™ 500
 - 3. Model 500 Regulatory Classifications Defined
 - 4. Symbols Used on the DANNIFLEX™ 500

- B. Introduction to Servicing the DANNIFLEX™ 500 pg. 15-16
 - 1. Voiding of Warranty
 - 2. Abbreviations Used
 - 3. Familiarity Statement
 - 4. Diagram Showing Major Parts of Unit

- C. Service Level I pg. 16-19
 - 1. Qualifications and Equipment Needed for Service Level I
 - 2. Periodic Maintenance for Service Level I
 - 3. Maintenance Instruction Level I
 - A. Knobs
 - 1. Removal
 - 2. Installation
 - B. Labels, Pads, & Velcro
 - 1. Removal
 - 2. Installation
 - C. Foot Assembly
 - 1. Removal
 - 2. Installation
 - D. Kneepot Cover
 - 1. Removal
 - 2. Installation

- D. Service Level II pg. 19-31
 - 1. Qualifications and Equipment Needed for Service Level II
 - 2. Periodic Maintenance for Service Level II
 - 3. Maintenance Instruction Level II
 - A. Bottom Cover
 - 1. Removal
 - 2. Installation
 - B. Cradle
 - 1. Removal
 - 2. Installation